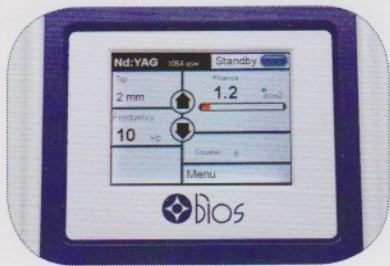
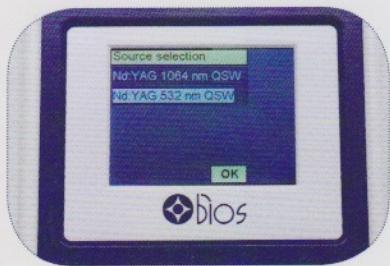
## BIOQS SERIES

Q-SWITCHED, SINGLE OR COMBINED  
WITH RUBY, LASERS

► High-tech laser and gold standard performances.

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**3**

**4**

**[1]** BIOQS Series is a range of laser devices based on a Q-switched Nd:YAG (532 and 1.064 nm) in combination with a Q-switched Ruby (694 nm), emitting high power pulses lasting nanoseconds and absorbed by the superficial epidermal pigments like stains or tattoos. Three wavelengths for multiple, effective and faster treatments.

**[2]** BIOQS SERIES delivers laser beams to the epidermis faster than the relaxation time of the healthy tissues in order to promote safety and efficacy, due the high selectivity, with no need of external cooling device.

**[3]** BIOQS SERIES laser beam selectively target the hyper pigmented lesions or the ink particles; the laser photo-acoustic interaction with the pigment causes the destruction of the pigment itself that is then eliminated by the lymphatic system. The surrounding tissues can't be damaged due to the high selectivity of the beam and to the different thermal relaxation time of the structures; there is no dispersion of heat and so no damages to the skin.

**[4]** BIOQS SERIES excels for the high Q-switched power, 1 J at 1064 and 694 nm and 450 mJ at 532 nm, that allows practitioners to work with a spot size up to 6mm, which means better penetration of the laser beam and so unexpected results, especially on deep tattoos

**[5]** BIOQS SERIES is nowadays even more used due to the "new" medical cosmetic and aesthetic applications motivated by trends and fashion: tattoos removal and permanent make up correction and removal guarantee a high regular number of new patients per month. The demand of tattoos removal, in particular, is incredible increasing year by year

**[6]** BIOQS SERIES guarantees a high return on investment due to the fast, effective, almost no bleeding and no permanent side effect treatments

**[7]** BIOQS SERIES is suitable for any skin type, from I to VI

**[8]** BIOQS SERIES is ideal for the following treatments:

- MULTICOLOUR TATTOO REMOVAL: black, blue, green and red colour
- BENIGN DERMAL AND EPIDERMAL PIGMENTED LESIONS: brown and age spots, solar lentigo, cafè-au-lait lesions, melasma
- ONYCHOMYCOSIS
- WARTS
- NON ABLATIVE SKIN RESURFACING: wrinkles and acne scars

**[9]** BIOQS SERIES could be equipped, upon request, with an optional IPL handpiece, with interchangeable filters, and integrated cooling system to extend the range of treatments carried out at the clinic

**[10]** BIOQS SERIES could be equipped, upon request, even with Er:YAG laser (2.940 nm) handpiece to perform skin rejuvenation, fractional ablative rejuvenation and surgical dermatology

BIOQS SERIES is highly appreciated by both doctors and patients because of the:

- Minimized downtime
- No permanent side effects
- High comfort during the treatment
- Fast and effective treatments
- Safe and long lasting results

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Bios S.r.l. reserves the right to make changes on the technical product data.
Bios S.r.l. si riserva di apportare modifiche ai dati tecnici del prodotto.
Bios S.r.l. se reserva el derecho de modificar la información técnica del producto.
### Bios S.r.l.